

# Raucodrape<sup>®</sup> Surgical Incise Drape



Incision site is clearly visible without glare



Provides protection from laterally migrating bacteria



Tear resistant, yet can be cleanly cut with a scalpel



Very conformable on awkward operation sites



Provides a sterile working field, without fear of contamination

# Raucodrape<sup>®</sup> Surgical Incise Drape

## Product Composition

Polyurethane film coated with polyacrylate adhesive

## Features

### Skin-friendly adhesive

- Elastic and conformable
- Bacteria-proof
- Permeable to water vapour
- Transparent and glare-free
- Red holding edges for user-friendly application
- Raw materials free of natural rubber latex

## Benefits

- The incision site is clearly visible without glare.
- **Raucodrape<sup>®</sup>** Surgical Incise Drape provide protection from laterally migrating bacteria because it stays in place even throughout major operations involving prolonged retraction of wound edges.
- **Raucodrape<sup>®</sup>** Surgical Incise Drape is tear resistant, yet can be cleanly cut with a scalpel.
- **Raucodrape<sup>®</sup>** Surgical Incise Drape is very conformable on awkward operation sites, yet its elastic qualities make it suitable for use where heavy manipulative surgery is involved, for example, in a total hip replacement.
- **Raucodrape<sup>®</sup>** Surgical Incise Drape provides a sterile working field, on to which internal organs may be placed without fear of contamination.

## Ordering Information

Sterile, Individually sealed in a Special pouch.

Adhesive Surface (cm)	REF	Shipping Units (pcs)
15 x 20	25441	10/60
30 x 20	25442	10/80
40 x 35	25444	10/80
45 x 50	25445	10/80
45 x 65	25446	10/40

## Indications

### Suitable for all types of Surgery:

- Orthopedic Surgery
- Cardiac Surgery
- Thoracic Surgery
- Gynecological Surgery
- Major abdominal Surgery
- Neurological Surgery
- Ophthalmic Surgery
- Pediatric Surgery

## Instruction for Use

Disinfect and dry the field of operation. Cover the patient. Apply only to perfectly dry skin. If, for lack of time, skin is not yet dry, then use sterile swabs to dry the skin before applying the surgical incise drape.

1. Open the peel-bag. Remove the film under sterile conditions.
2. Unfold **Raucodrape<sup>®</sup>** over the field of operation and hold at the red edges.
3. Peel off the cover paper parallel to the holding edge.
4. Hold **Raucodrape<sup>®</sup>** crease-free and, starting from the centre, smooth on to the skin with a sterile cloth.
5. Remove **Raucodrape<sup>®</sup>** surgical incise drape from the patient's skin by lightly pulling at the holding edges and overstretching the film.
6. Do not remove **Raucodrape<sup>®</sup>** incise drape from the skin at an angle of 90°.

**Raucodrape<sup>®</sup>** adheres firmly even during prolonged and major operations. Sterile unless the package is opened or damaged.



[www.covidsafeindia.com](http://www.covidsafeindia.com)



**Jeevan Jagga**  
Founder & CEO  
+91 98103 01515  
[jeevan@covidsafeindia.com](mailto:jeevan@covidsafeindia.com)

**COVID**  
SAFE INDIA